



For support: www.imageanalysisgroup.com

Email: support@imageanalysisgroup.com

Call: +44 (0) 7413022053

Release Notes


Dynamika Enterprise Hotfix version 5.0.6

21st Apr. 2017

New Features

- Added relaxivity values in blood plasma to lookup table for Pharmacokinetic Analysis
- Added missing units to graphs and tables

Electronic Instructions for Use

- The user manual is available in the application by clicking the help button  on any page after login and selecting “Download user guide in PDF” or on the web <http://www.imageanalysisgroup.com/dynamika-online-resources>

Data Security

- Dynamika’s cloud-based edition is only deployed on service providers with state-of-the-art data security and data encryption technology
- It is the responsibility of the user to manage access to any local installation of Dynamika as DICOM data in the local data storage container is unencrypted. It is recommended to use hard-drive encryption.

Virus Protection

- Every effort is made to ensure all Image Analysis software is virus free
- It is recommended that all computers used to run or access Dynamika Enterprise have up to date virus protection

Licences

- To receive licences to run the software please contact support – see contact details at the top of page

System Requirements

- Dynamika Enterprise™ is a web based application that can be installed on a server and accessed from client computers via a web browser. It is possible to install the server components on a personal computer and use Dynamika as a ‘Workstation’.
- Installation of the server side software is to be conducted by qualified, trained Image Analysis IT personnel, only.

Server requirements

- Operating System: Linux/Windows Server 2003 (Ubuntu Linux preferred)
- Processor: minimum Dual Core (8 recommended)
- RAM: minimum 4GB (8-16GB recommended)
- Disk: Enough to store all images, minimum 50GB



For support: www.imageanalysisgroup.com

Email: support@imageanalysisgroup.com

Call: +44 (0) 7413022053

Client requirements

- Processor: Intel Core i3 (Intel Core i7 recommended) or comparable
- RAM: minimum 4GB
- Minimum screen resolution: 1024x768

Browser Requirements

- Chrome v50 or later – recommended using latest stable release
- Firefox v45 or later
- Safari v9 or later

PC/MAC requirements (as Workstation)

- PC Operating System: Windows 7, 8.x, 10,
Linux Ubuntu 12 or later, Debian 7 or later
- MAC Operating System: MAC OS 10.9 or later
- Processor: minimum Intel Core i3 dual-core (Intel Core i7 with 8 cores recommended) or comparable
- RAM: minimum 4GB (8-16GB recommended)
- Disk: Enough to store all images, minimum 50GB

Thank you for using DYNAMIKA!

We look forward to hearing your feedback and suggestions

Dynamika, Dynamika Enterprise, Dynamika Workstation and Dynamika-RA are trademarks of Image Analysis Ltd ©2009-2017